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**INTRUCTIONS, ASSEMBLING
AND DISASSEMBLING
INSTRUMENTS**

**INSTRUMENT CLASS I
USE:**

• UTERINE MANIPULATOR

FOR THE DISASSEMBLING:

- A) Take off the tip (1), unthreading it from the cup (2), spinning until it loosens.
- B) Take the cup off (2), unthreading it from the terminal (4), spinning until it loosens.
- C) Completely remove the seal (3) from the instrument.

FOR THE ASSEMBLING:

- D) Put the seal (3) back on, inserting it to its place of origin.
- E) Put the cup back on (2), threading it to the Terminal (4), spinning until it tightens.
- F) Put the tip back on (1), threading it to the cup (2), spinning until it tightens.

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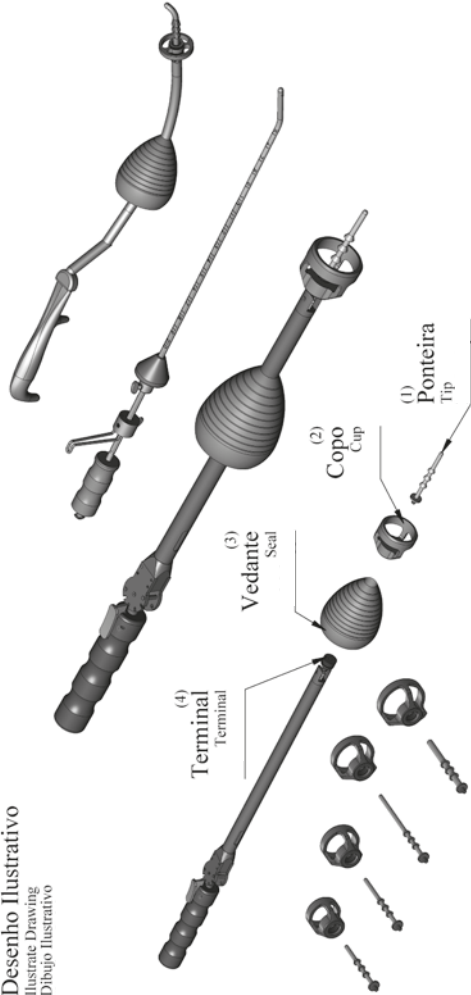
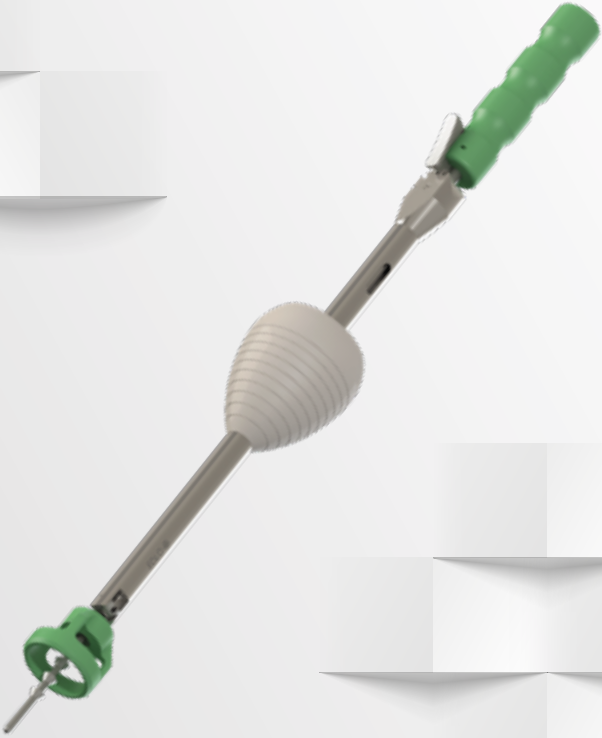
MANIPULADOR UTERINO
UTERINE MANIPULATOR
MANIPULADOR UTERINO

FRÁGIL
FRAGILE
FRÁGIL

MANTER SECO
KEEP DRY
MANTENGALO SECO

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SURGICAL INSTRUMENTS GENERAL CONSIDERATIONS

ATTENTION Read all information. Misuse and improper handling cause premature wear and decrease the instrument's life. This equipment must, obligatorily, be used sterile and handled by qualified personnel: physicians and nursing professionals.

FIRST TIME USE Before the first use, the instrument must be cleaned and sterilized. EDLO video laparoscopic surgery instruments are compatible with the Autoclave and ETO sterilization methods, (See sterilization methods).

FUNCTIONAL INSPECTION AND VERIFICATION It is extremely important to carefully examine each and every instrument for breakage, cracks, insulation damage or malfunction before use. Special attention should be paid to the jaws and articulations and mobile parts. DO NOT USE DAMAGED SURGICAL INTRUMENTS Never try to repair the instruments. Maintenance and repairs should be done by qualified

personnel only. Questions concerning repairs should be sent to your region's Business Consultant or directly to the address mentioned in this Manual.

DECONTAMINATION, CLEANING AND MAINTENANCE Every instrument must be disinfected and cleaned after each use. Carefully clean and inspect each instrument. Sterilize before use. Appropriate cleaning and maintenance prolong the instrument's life. Instruments must be dismantled during cleaning and sterilization.

INSTRUMENTS FOR ELECTROSURGERY Once you have started using Electrosurgery instruments, some special safety measures must be taken into account, so that severe risks due to improper use can be prevented. Contact with non-insulated instruments must be avoided. Make sure the instrument is well connected to the cables and power generators. Make sure the grounding plate is duly connected. 1KHZ and 250V limits must be respected.

CARE

- Do not use corrosive chemical products to sanitize.
- Cleaning and washing solutions must have a neutral pH factor (7.0).
- Use enzymatic detergent during cleaning.
- Do not use abrasive cleaning solutions.
- Use only soft bristle brushes.
- Ultrasonic washers may be used.
- Use water as dictated by ABNT 11816-2003 or EN-285.
- Rinse with distilled, deionized or demineralized water.
- Dismantle instruments in preparation for sterilization.
- After washing, dry instruments completely with compressed air.

STERILIZATION METHODS Steam autoclave sterilization is recommended. Standard cycle for autoclave: 134º for 5 minutes. Different times and steam temperatures can be used. However, user must validate any and all recommended time and temperature variances. The temperature must not exceed 137º (288ºF). The ETO sterilization method can also be used. The parameters for using this process follow the standard.

HANDLING The surgical instruments must be carefully handled, transported and stored in moisture-free boxes. This measure is especially important for jaws and articulations. Heavier, sturdy instruments should not be placed on top of more delicate instruments.

WARRANTY All EDLO products have a 1-year warranty against raw-material, manufacturing or transportation defects. EDLO products will no longer be covered by the warranty if they are modified, mishandled, misused, or used for purposes not in accordance with what they were made for.

CONTACT YOUR SALES REPRESENTATIVE OR THE FACTORY FOR REPLACEMENT PARTS.

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