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CLAMP WITH ELECTRICAL CONNECTION

WARNING: Read all the information. Incorrect handling and operations lead to premature wear and decreased useful life for the instrument. This device must be used sterilized and by qualified personnel: physicians and nursing professionals.

FIRST USE: Before the first use, every instrument must be cleaned and sterilized. EDLO laparoscopic surgery instruments are compatible with the autoclave sterilization method. (See sterilization).

INSPECTION AND FUNCTIONAL CHECK: It is important to check thoroughly every surgical instrument for breaks, cracks, insulation damage or malfunctioning before use. Special care should be given to areas such as the jaws, as well as all movable parts and joints. Do not use damaged surgical instruments! Never try and repair instruments. Servicing and repair must be performed by trained and qualified personnel. Repair questions should be referred to the Business Consultant in your area or directly to the address mentioned in this manual.

ELECTROSURGERY: Video-assisted surgery clamps have an electrical connection for monopolar procedures. Proper caution must be taken when handling and operating so as to prevent unnecessary damage to the patient, the user and the product. Before use, check the connections and the adjustment of the power generating equipment.

Connect the clamp to electronic scalpels that do not exceed the technical characteristics described in the table below.

TECHNICAL SPECIFICATION AND CHARACTERISTICS:

ANVISA Hazard Class	Class III
Protection against electrical shock	Type BF, Class I Device with an Applied Part
Maximum working power	From 150 W to 2.1 KV and 400 W to 1.8 KV
Maximum frequency	390KHz

FITTINGS: For connection to the clamp, use the appropriate cable with standardized connectors; however, according to availability, adapters can be used for a safe and effective connection. These include:

- Male/female monopolar cable; Male/male monopolar cable; Female/female adapter; Male/male adapter.

DECONTAMINATION AND CLEANING: All video-assisted surgery clamps must be disassembled, cleaned, sterilized and reassembled for use. The assembling and disassembling procedure is presented in these Instructions for Use with the description of the sequence and a drawing showing the correct position of the parts.

Dgtech. Ind. e Com. Ltda. has a validated procedure for cleaning and disassembling the clamp by using brushes with soft bristles and enzymatic detergent for brushing, including brushes suited for cannulated parts. A standard 30-minute cycle is used in an ultrasonic washer, followed by rinse with water with quality of reverse osmosis.

Subsequently, drying is performed in a dryer at the temperature of 55°C for 30 min. Pack the disassembled parts for sterilization.

STERILIZATION: Dgtech Ind. e Com. Ltda. uses saturated steam under pressure in a cycle of 134°C for 5 min as its validated sterilization process.

We stress that the use of this method must be validated by the facility that will apply it, thus ensuring the sterilization of the product.

PRECAUTIONS:

- Do not use corrosive chemicals for cleaning;
- The most appropriate cleaning and rinsing solutions should have neutral pH (7.0);
- During cleaning, use enzymatic detergents;
- Do not use abrasive cleaning solutions;
- For brushing, use brushes with soft bristles only;
- Use ultrasonic washers;
- Use water in accordance with the NBR11816 or EN285 standards;
- Rinse thoroughly with reverse osmosis water;
- Prepare for sterilization with the instruments disassembled;
- After washing and rinsing, dry the instruments completely and carefully using compressed air.

HANDLING: Surgical instruments must be handled carefully, carried and stored within boxes and be free of moisture. This is important especially for jaws and joints. Robust instruments must not be put on top of delicate instruments.

TECHNICAL ASSISTANCE: All products manufactured by Dgtech Ind. e Com. Ltda. must be repaired in the factory only. Use the address included in these Instructions for Use to send the products for servicing or warranty. Leave them in the care of the Technical Assistance department. The products sent to technical assistance must be clean and sterilized and may be returned if they are not in this condition.

ENVIRONMENTAL PROTECTION: At the end of the useful life of the instrument, the following risks are associated:

- Contamination;
 - Improper reuse causing risk due to malfunctioning, this may harm the patient and the user.
- In order to mitigate risks, the device must be disposed of at an appropriate site in accordance with the current legislation, such as that for hospital waste. They may also be sent to the manufacturer for disposal as long as properly clean and sterilized.

WARRANTY: All EDLO products have a 1-year warranty for raw material and workmanship defects or defects caused during transportation. EDLO products are not under warranty when they are changed, showing characteristics of misuse, or when they are not applied according to the purpose for which they were designed.

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